

Setting Up Your CD Drive

1. Start the program.
2. Select **Interface** from the **Configure** menu.
The **Interface Configuration** dialog box is displayed.
3. Create the CD drive configuration.
 - In the **Device Name** text box, enter a name for the drive.
 - From the **Drive Name** drop-down list, select a drive letter (automatically detected by the personal computer).
 - From the **Device Type** drop-down list, choose a type of functionality.
 - For read/write functionality, choose **CDR Multi Session**.
 - For read-only functionality, choose **CD ROM**.
4. Click **Add** to update the table with the new CD drive configuration.
5. Click **OK** to save the new configuration and close the dialog box.

Viewing Images from CDs

1. Insert the CD into the CD drive on your computer.
2. Start the program and then click the **Open Patient Browser** button to display the patient browser.
3. Select the CD drive icon on the left of the **LOCAL** tab to display patient data stored on the CD.
Note: If the patient data on the CD is not displayed, then ensure that the CD drive is set up for operation with the program.
4. To view images, double-click the required patient, study, or series from the **PATIENT LIST** on the right of the **LOCAL** tab.
The program closes the patient browser and displays the images in the **Review** task card.
5. For software versions below 2.0: To reduce the time required for volume rendering, first save the loaded images to the database (computer's hard disk)—select **Save to Database** from the **File** menu.
6. To view a volume or sequence of volumes, select the related image in the **Review** task card and then click the **Compose** task card tab.